+971 50 713 3923

Graphy

Shape Memory
Aligner®
Direct Aligner®

GRAPHY PRACTITIONERS INSIGHT ALIGNERS AND ALLERGIES

The intent of this article is to highlight a potentially under-reported effect of newer materials commonly used in orthodontics. With growing evidence of allergic incidents linked to aligner materials, general practitioners (GPs) and orthodontists must take steps to protect their patients—and themselves—from potential medicolegal issues. To date, no official protocols have been proposed. To raise awareness among both patients and clinicians, we propose the following:

- Maintain detailed case records, including a comprehensive allergy history, especially regarding delayed hypersensitivity reactions.
- If necessary, perform a patch test for diaminodiphenylmethane (MDA), hexamethylene diisocyanate (HDI), or other potential allergens (e.g., isocyanate groups) before prescribing aligners.
- Include specific statements about potential allergic reactions in the patient's informed consent form.
- Educate patients on the possibility of background exposure to MDA and HDI, which may contribute to allergic reactions.
- Record baseline allergy indicators (e.g., ESR) at the time of each new aligner insertion/removal.
- For high-risk patients, initial screening may include patch testing for dental and isocyanate series.
- Carefully document all abnormal health changes and related interventions in the case records.
- Educate patients about preventing allergic episodes and outline protocols for reporting and managing acute allergic reactions—including use of an emergency medical card.
- Train allergy-prone patients to recognize the signs and symptoms of potential reactions and seek appropriate medical help

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